
# How to Apply WoundSeal MD

**For Nosebleeds:** Use only in the lower part of the nasal cavity. Gently blow nose to clear clots and apply only when bleeding resumes. WoundSeal MD needs blood to work. Do not tilt head during application. Avoid inhaling while applying WoundSeal MD Powder to avoid sneezing and coughing.

**1. Load Applicator** by rolling it in powder.

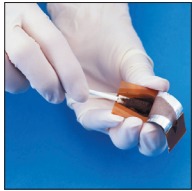
**2. Roll Applicator** inside the nostril to coat with powder.

**3. Remove Applicator and Pinch Nose** for 15 seconds.

**For Headwounds and other hard-to-reach areas:** Use enclosed applicator to apply powder.

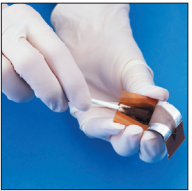**1. Peel** back of blister then load applicator.

**2. Cover Wound** completely with powder.

**3. Press** for 30 seconds to form the protective scab and stop the bleeding.

Occasionally upon application of Powder a temporary warming or stinging sensation may be felt. Repeat application as needed. The protective scab may be left in place until it falls off naturally. Avoid wetting the scab for one hour. If WoundSeal MD Powder must be removed as determined by a healthcare professional, brush away loose powder then soak wound with sterile water until scab starts loosening from wound. Gently scrape off loosened scab with gloved finger or tongue depressor.

**Warnings: FOR EXTERNAL USE ONLY.** Not intended to replace stitches. Seek medical attention if wound requires stitches. Do not apply WoundSeal MD Powder to visibly infected wounds. Keep out of reach of children. May be harmful if inhaled or ingested. Avoid aspirating the product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Seek medical attention if any irritation persists or if bleeding continues as this could indicate a more serious problem.

**CAUTION:** Store in a clean, dry place. Use by the expiration date listed. Discard if dispenser is damaged, opened, or if powder has caked and does not flow freely. Discard contents after use, as WoundSeal MD Powder is guaranteed effective only if it has not been left exposed to air or humidity.

**Ingredients:** Hydrophilic polymer and Potassium Ferrate.

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